## dental diamant

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>>>

## dental diamond

### Directions for use of rotary diamond instruments (FG and right angle type)

The following instructions enable the best preparation results to be achieved by handling and using the instruments carefully. Of paramount concern are conservation of the tooth substrate and hygiene requirements.

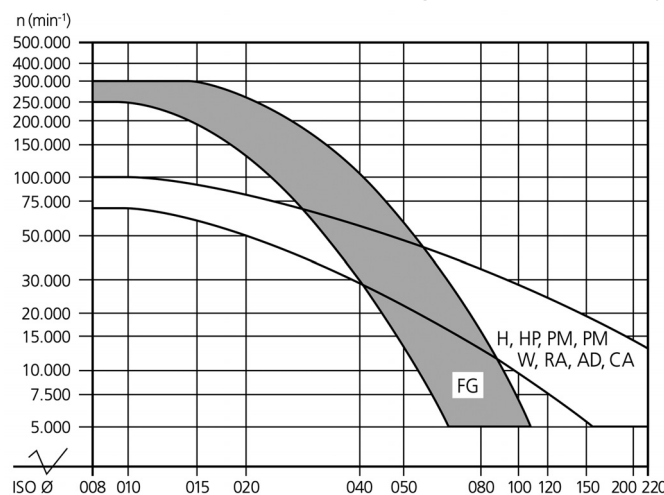
**Intended use:** Use in the handpiece of a micromotor or turbine in the dental practice. Grinding of tooth substance such as tooth enamel or dentine, but also smoothing of new ceramic or composite fillings and removing of old ceramic or composite fillings.

**Intended users:** dentists

**Incidents:** All serious incidents related to the dental diamond burs must be reported to the manufacturer and to the authority of the Member State where the user and/or patient is established.

- The separate information and instructions for cleaning and sterilising electroplated dental diamond burs with FG or right angle shanks apply with regard to cleaning/sterilisation.
- Instruments are to be selected according to their diameter and shape so as to avoid damaging adjacent anatomical structures.
- Preference should be given to rounded shapes.
- The indication is limited for instruments with coarse grit; as finish grinding is usually required.
- It is advisable to use drill holders or, for longer storage, the original packaging.
- Make sure the instruments are firmly secured when using them in a nozzle or handpiece.
- The speeds given in the diagram, taking into account the head diameter, are not to be exceeded. Bring the instruments up to working speed away from the preparation area.
- Spray cooling with a minimum flow of 50 ml/min is to be used to protect the dental pulp.
- Additional cooling is required when using instruments longer than 21 mm or with a head diameter larger than 2 mm.
- The force must not exceed 2 N (200 p); a greater force will increase the thermal load, but will not remove the material more quickly.
- Worn, damaged or non-concentrically running instruments must be disposed of.

- The instruments are to be placed in a "drill bath" between being used and sterilised (follow the manufacturer's instructions).
- A hard brush can be used for mechanical cleaning of the grinding layer.
- Diamond instruments are resistant to corrosion; however, they must not be exposed to acids.

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Permissible speed depending on the head diameter

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## Information and instructions for cleaning and sterilizing electroplated dental diamond burs with FG or right-angled shanks.

Here you will find recommendations and instructions for handling the above-mentioned electroplated diamond tools.

### 1. Products

In general, fis dental diamond burs with FG or right-angled shanks can be reused, provided they are reprocessed before second use. Used burs with FG or right-angled shanks must be cleaned, inspected and sterilized before their next use. New burs from the factory must be sterilized before first use.

### 2. Qualification of staff reprocessing these products

This work should only be carried out by someone who has gained the necessary knowledge and skills through training, qualifications or experience.

### 3. Training

When using new products or updating/changing procedures, additional training is required.

### 4. Further documents

It must be ensured that the recommendations of the Robert Koch Institute and the German Federal Institute for Pharmaceuticals and Medical Products in "Requirements for hygiene when reprocessing medical products" are followed, especially those concerning dental health.

### 5. Transport and storage

The products are supplied unsterilized. As the burs have a rough surface, cleaning and disinfecting requirements are particularly high. The tools must be stored and transported in such a way as to avoid damage to the high-precision shanks by the grinding layer. To avoid grinding residue hardening, the tools must be put into a suitable cleaning bath or disinfectant (drill bath) directly after use (within 60 minutes).

## 6. Cleaning and disinfection

### 6.1 Pre-treatment at place of use

First, the tools should be cleaned with a lint-free disposable cloth.

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## 6.2. Disposal and transport to area of reprocessing

We recommend putting the tools directly into a cleaning bath or disinfectant (drill bath) after use and transporting them in the fluid to the area of reprocessing.

## 6.3. Manual cleaning and disinfection

- The grinding layers of the dental bur should be rinsed thoroughly under running cold water and cleaned with a hard brush (not a metal wire brush) to remove loose particles as well as residue from cleaning agents or disinfectant. The brush should then be cleaned and disinfected.
- Clean the burs in an ultrasonic bath (0.5% neodisher® MediClean forte) at a temperature of <40° Celsius and a frequency of 35kHz for 10 minutes. Please refer to the recommendations of the bath and cleaning agent manufacturers.
- To avoid damage while cleaning, the burs should not come into contact with the shanks of other tools.
- Clean products thoroughly with a soft brush and rinse under running drinking water (>1 minute).
- Dry manually using a lint-free disposable cloth or blow with sterile, oil-free compressed air.
- Dip products into a disinfectant listed by the RKI or the Association for Applied Hygiene (VAH). Follow the manufacturer's instructions. Ensure that the disinfectant reaches all areas of the product.
- The process is validated with the following disinfectant: 3% Korsorex Plus for 15 minutes. Alternatively use disinfectants listed by VAH ([vah-online.de/en/vah-list](http://vah-online.de/en/vah-list)), e.g. 100% ID 220 for 1 minute.
- Rinse the product with de-ionised water for at least 1 minute (or as instructed by manufacturer).
- Dry with a lint-free disposable cloth. We recommend blowing the grinder with sterile, oil-free compressed air.
- Do not heat product to more than 140°C.

- After cleaning in an ultrasonic bath, inspect the bur with a magnifying glass with at least 6x magnification. If any particles or residue are still on the bur, the above steps must be repeated. If the grinding layer is worn, damaged or particles or residue cannot be removed, dispose of the tool. >>>

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This also applies to any damage on the shank or bur seen by the naked eye.

#### 6.4. Automatic cleaning and disinfection

Following guidelines from the RKI automatic cleaning should be the preferred method to ensure high reproducibility. Follow the manufacturer's instructions.

- The grinding layers should be rinsed thoroughly under running cold water and cleaned with a hard brush (not a metal wire brush) in order to remove loose particles and residue from cleaning agents and disinfectant.

- Place carefully into a disinfector.

The "Vario TD AD" process of a disinfector such as the Miele G 7835 CD is described below:

- 2 x rinse for 1 minute and then for 3 minutes with cold (<math><40^{\circ}</math> C) tap water. Water is then drained.
- Clean with tap water and alkaline cleaner (e.g. „neodisher® Mediclean Dental“ 0.5%) at  $55^{\circ}$  C for 10 minutes.
- Neutralization with cold (<math><40^{\circ}</math> C) tap water and 0.1% „neodisher® Z“ for 3 minutes.
- Rinse with cold (<math><40^{\circ}</math> C) deionised water for 2 minutes.
- Drainage of water and rinse with deionised water for thermal disinfection at  $92^{\circ} \pm 2^{\circ}$  C for at least 10 minutes.
- Drying cycle of 30 minutes at  $60^{\circ} \pm 5^{\circ}$  C.

The product may not be heated to more than  $140^{\circ}$  C

To avoid damage while cleaning automatically the burs should not come into contact with the shanks of other tools.

After cleaning in a disinfector inspect the bur with a magnifying glass with at least 6x magnification. If any particles or residue are found on the bur, the above steps must be repeated.

If the grinding layer is worn, damaged or particles or residue cannot be removed, dispose of the tools. This also applies to any damage on the shank or bur seen by the naked eye.

#### 6.5. Sterilization

All usable burs with diamond layers should be packed in suitable sterile packaging (ISO 11607) which has been steam sterilized in a fractional vacuum process at  $134^{\circ}$  C. The device should comply with EN 285 / EN ISO 17665-1 validated processes.

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- Fractional vacuum (4-fold)
- Sterilization temperature 134° C
- Dwell time 5 minutes (full cycle)
- Drying time 20 minutes

When stored, the tools must be protected from contamination.

### 7. Return of dental diamond tools

In case of a complaint, the dental diamond burs are withdrawn. Should the tools have been removed from the packaging or should they have been used, we require them to be cleaned and sterilized according to our instructions before returning them. Otherwise, the dental diamond burs shall be packed as contaminated and marked accordingly. In the download section of our website we provide a corresponding form ([www.fis-online.com](http://www.fis-online.com)).

### 8. Concluding remarks

Applicable national regulations are to be observed when reprocessing used diamond tools.

On account of their materials and workmanship, tools manufactured by fis are generally suitable for reprocessing using the described procedure. Users are responsible for reprocessing the tools in compliance with this recommendation and the operating instructions and instructions for use from the manufacturers who supply the appropriate devices and materials. Users are also responsible for ensuring that the reprocessing attains the intended results with the described equipment and qualified personnel.

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### Legende

On our packages the below listed pictographs will appear to indicate the specific use:

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